

## Broken screw extractor protocol




1- Visualize the embedded broken shank.



2- Select the centring guide according to the connection and platform.



3- Place the centering guide over the implant using the wrench.

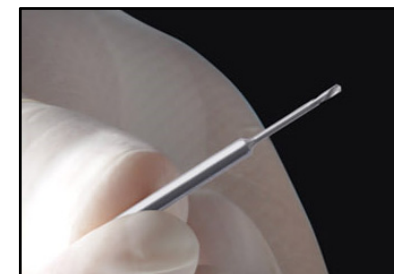

4- Select the FEX100 drill.



5- Place the drill in the contra-angle.



6- Insert the drill directly into the centering guide with the contra-angle.




7- Remove the guide and drill the shank with the contra-angle.



8- Select the extraction punch.



9- Insert the punch into the shank.

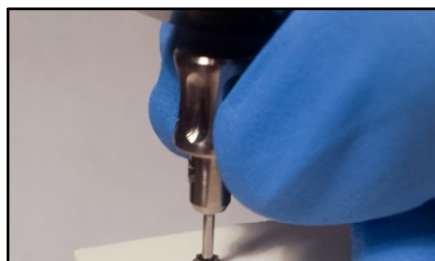

10- Hit the punch with the help of a hammer.



11- Remove the shank.

