

# IMPLANT PACKAGING

IFU - IMP - ANNEX (Edition 4: 01 - 10 - 2021)



1. Open the outer implant box using the side tab.



2. Remove the blister containing the implant..



3. Remove the protective cover from the blister using the designated pull tab.



4. The protective cover contains 2 labels: one must be placed in the patient's implant passport and the other in the clinical record.



5. Remove the implant container from the blister. The plastic cap incorporates the implant cover screw.



6. Place the plastic implant carrier onto the square connection of the metal implant carrier.



7. Remove the implant by lifting it vertically upwards.



8. It is also possible to carry the implant to the mouth with the contra-angle by inserting it directly into the metal implant carrier.



**MICRODENT**