

IMPLANT PACKAGING with multifunction abutment

IFU - IMP- PMF - ANNEX
(Edition 3: 01 - 10 - 2021)




1. Open the outer implant box using the side tab.



2. Remove the blister containing the implant.



3. Remove the protective cover from the blister using the designated pull tab.

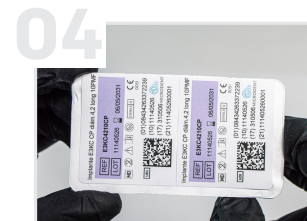

4. The protective cover contains 2 labels: one must be placed in the patient's implant passport and the other in the clinical record.



5. Remove the implant vial from the blister. The plastic cap incorporates the implant cover screw.



6. Remove the cap from the vial.



7. The implant can then be removed using the manual wrench for multifunction abutments.



8. It is also possible to remove the implant using the contra-angle driver for multifunction abutments.



MICRODENT