


IMPLANTS without implant-holder PACKAGING



1. Open the implant threaded cap counter-clockwise.

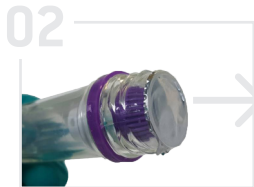


2. Remove the aluminum cap from the container by a flap.



3. Once the cap has been removed, extract the implant's container. The plastic cap contains the cover screw.



4. Remove the plastic cap from the inner container.

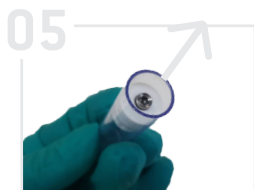

5. Once the plastic cap has been removed, it is ready for extraction.



6. Then, remove the implant with the direct key to the implant, connected to the contra angle.

Instructions for use
**IMPLANTS without
implant-holder PACKAGING**



MICRODENT
IMPLANT SYSTEM