


MONOBLOCK IMPLANT PACKAGING

IFU - IMP - WE - ANNEX
(Edition 1: 01 - 10 - 2021)

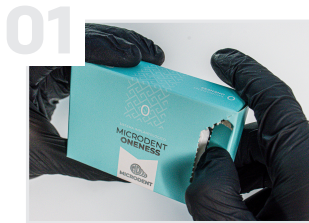

1. Open the outer implant box using the side tab.



2. Remove the blister containing the implant.



3. Remove the protective cover from the blister using the designated pull tab.



4. The protective cover contains 2 labels: one must be placed in the patient's implant passport and the other in the clinical record.



5. Remove the implant vial from the blister.



6. Remove the one-piece implant by lifting it vertically upwards.



7. Optional: The one-piece implant may be separated from the assembly cap to facilitate insertion in teeth in the posterior region.



MICRODENT